


DrugWipe® 3S

Type Approved in GB


Mobile and Police Station preliminary drug testing device for: Cannabis, Cocaine.

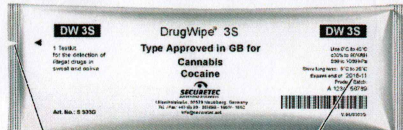

Tear open at cut. Expiry date.

- Supervise or check that subject has an empty mouth and has not eaten or drunk anything for 10 minutes prior to the test.
- Put on a new pair of sterile gloves if required.
- Check "expiry date" & there is no damage to packet.
- Tear open packet remove DrugWipe® 3S
- Check read out window has faint blue lines (see step 12 Unused) discard if any red lines are present.

- Slide grey cover to first click. Hold gently in finger tips. Remove blue sample collector.

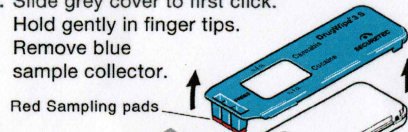


Red Sampling pads

Caution:

Do not place "pads down".
Do not touch sampling pads.

- Whilst observing, ask subject to rub tip of tongue along very bottom of left cheek 3 times, then bottom of right cheek 3 times and then stick their tongue out. Stroke blue sampler along tip of tongue, first down left, then down middle, then down right.

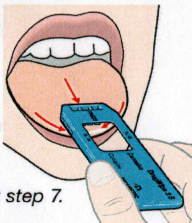

Caution: Look for saliva or repeat step 7.

© D.tec 12/2014

- Place sample collector over and check for colour change on the three pads from red to yellow.

Sampling pads must turn yellow




Caution:

If sampling pads not yellow, repeat steps 7&8

- Replace blue sample collector on to the white body. Close firmly with a double click.

Note:
check again it is firmly closed.

Click!
Click!

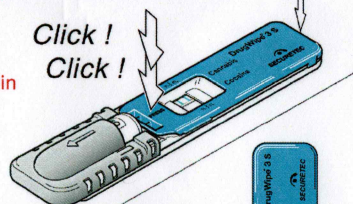

- Start timing 8 minutes. Hold vertically, blue up. Slide down grey cover. With tip of thumb, PRESS several times and break ampoule completely. Count to 10 seconds. Slide grey cover back up.



10 Seconds

- Place on level surface eg. Diagram in step 12.


STORAGE, TRANSPORT, USE & DISPOSAL

Use DrugWipe..... 0°C to 40°C
Transport DrugWipe..... 0°C to 40°C
Short term store DrugWipe..... 5°C to 40°C
Long term store DrugWipe..... 5°C to 25°C
Do not freeze, over heat or leave in direct sunlight.
Dispose: General or Clinical waste if available.

www.dtecinternational.com - Phone 0800 371 898


- At 8 minutes, read results from result window.

Two red control lines indicate a valid test and additional red test lines indicate lab test required.

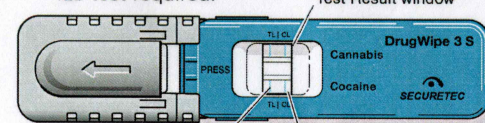


Test Line for individual drug on DrugWipe® 3S

BLUE Control Line, on Unused DrugWipe® 3S

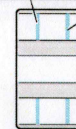

UNUSED.

Light BLUE Control Lines and Test Result Lines

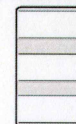

X

INVALID TEST.

No Red Control Lines

Action: Take a New DrugWipe® 3S and repeat all steps exactly.




VALID TEST.

Red Control Lines
No Red Test Result Lines
NO SPECIFIED DRUG DETECTED

(above analytical cut off)

Action: Consider presence of drugs other than those tested with this device.

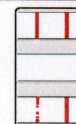

VALID TEST.

Red Control Lines
Red Test Result Line or Lines
SPECIFIED DRUG DETECTED

+

Action: Record result & prepare subject for blood sample as soon as possible.

Note: Un-even, partial or faint red Test Result Lines are SPECIFIED DRUG DETECTED. Ignore any changes 15 minutes after test result is read.